

Greatcare

**China medical
devices manufacturer,
exporter & importer**

Irrigation Set Bewässerung-Beutel Bolsos De La Irrigación

NOTE:

- For single patient use only.
- Sterilized with ethylene oxide
-
- Nur für einmaligen Patientengebrauch.
- Sterilisation mit Ethylenoxid

Ningbo Greatcare Trading Co., Ltd.

Unit 93, building 12, No.818 Qiming Rd.
Yinzhou, Ningbo, China.

Tel: +86-574-8778 6632 Fax: +86-574-8778 6644

E-mail: info@greatcare.cn



Creating a Better Care



- 1-1500cc Enema Bag
- Touch-N-Seal™ VALVE
- Prelubricated Tip
- Lube Cap
- Tubing Clamp
- 1-Moisture Proof Drape
- 1-Enema Soap packette
- 2- VINYL Gloves (optional)

SUGGESTED PROCEDURE

- Slide tubing clamp to "off" position.
- Place two fingers under blue pull tab, pull forward to open top of bag.
- Add desired quantity of selected solution.
- "Squeege" across the valve closure with thumb and forefinger to seal bag.
- Suspend bag for administration.
- Position moisture proof drape under patient.
- Remove lube cap from tip of tube.
- Proceed with administration using approved technique.
- Drain unused solution from bag and discard.